



MEDICAL ALERT QUICK START GUIDE

AtsCares.com

Step #1 - REVIEW RECEIVED CONTENTS

--White Base Unit, Pendant, Batteries

(2) Li-Ion Pendant batteries, (4) Base unit rechargeable batteries


--Clips/Lanyard (1) Pendant wrist clip, (1) Pendant belt clip, (1) Pendant lanyard

--Phone Cord Telephone Cord for connecting ATS Medical Alert System to a phone line

--Power Adaptor Base Unit power adaptor

Step #2

Install 4 batteries in the base unit. Remove cover on the bottom of the base unit and install batteries as shown on the unit

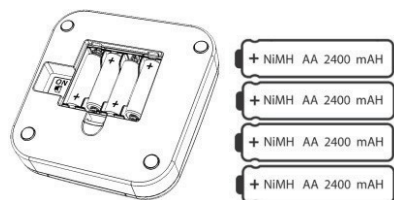


Step #3

Plug the power adaptor into back of base unit and into electrical plug in the wall.



Step #4

Charge the pendant battery. Install 1 pendant battery in the base unit charger.

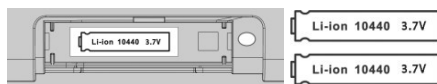

The LED on the right side indicates charging status:

- LED OFF - No battery installed
- LED RED - Battery is charging
- LED GREEN - Battery is fully charged



MEDICAL ALERT QUICK START GUIDE

AtsCares.com

Step #1 - REVIEW RECEIVED CONTENTS

--White Base Unit, Pendant, Batteries

(2) Li-Ion Pendant batteries, (4) Base unit rechargeable batteries


--Clips/Lanyard (1) Pendant wrist clip, (1) Pendant belt clip, (1) Pendant lanyard

--Phone Cord Telephone Cord for connecting ATS Medical Alert System to a phone line

--Power Adaptor Base Unit power adaptor

Step #2

Install 4 batteries in the base unit. Remove cover on the bottom of the base unit and install batteries as shown on the unit

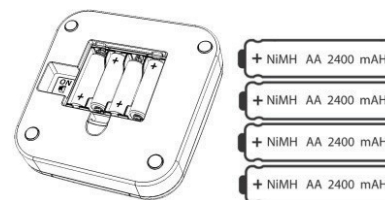


Step #3

Plug the power adaptor into back of base unit and into electrical plug in the wall.



Step #4


Charge the pendant battery. Install 1 pendant battery in the base unit charger.

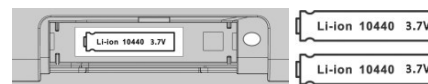

The LED on the right side indicates charging status:

- LED OFF - No battery installed
- LED RED - Battery is charging
- LED GREEN - Battery is fully charged

After 4 hours of pendant battery charging - take the other battery and place in the charger and place the 1st battery in the pendant.

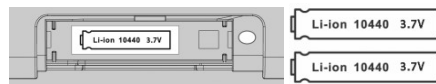


Step #6

Unscrew the battery cap on the pendant. Install the battery with negative side up (silver end) on top / facing cap. Screw cap back on the pendant.



Step #7

Connect the base unit to a telephone line.

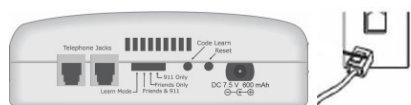


Step #8-PROGRAMMING CONTACT NUMBERS

- Using any telephone that is connected to the same phone line as the alert system, call a friend and tell them you are going to program your alert system and to please hold as you do it;
- Slide switch at back of the base unit to 'Learn Mode' (you will hear the base unit speak "Learn Mode" from the speaker);
- Enter "1 2 3 4 #" into the telephone keypad (you will hear the base unit speak the numbers 1, 2, 3, 4 and # as you press the buttons);
- Enter first contact number and press the # (you will hear the base unit speak the numbers as you press them);
- Enter second number followed by # (you will hear the base unit speak the numbers as you press them) if you do not have a second number, press#;
- Enter third number followed by # (you will hear the base unit speak the numbers as you press them) if you do not have a third number, press#;
- Enter fourth number followed by # (you will hear the base unit speak the numbers as you press them) if you do not have a fourth number, press#;
- You will hear voice prompt "Programming Complete" (if you do not hear "Programming Complete", you must start over at the beginning of this section again);
- Slide switch back to desired location of Friends Only, Friends and 911 or 911 Only.

For set-up assistance, to order extra pendants or wall communicators, call (615) 562-0043

After 4 hours of pendant battery charging - take the other battery and place in the charger and place the 1st battery in the pendant.

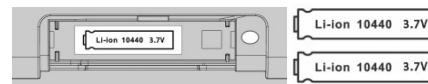


Step #6

Unscrew the battery cap on the pendant. Install the battery with negative side up (silver end) on top / facing cap. Screw cap back on the pendant.



Step #7

Connect the base unit to a telephone line.

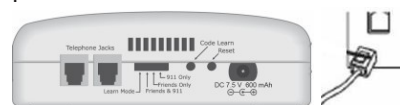

Step #8-PROGRAMMING CONTACT NUMBERS

- Using any telephone that is connected to the same phone line as the alert system, call a friend and tell them you are going to program your alert system and to please hold as you do it;
- Slide switch at back of the base unit to 'Learn Mode' (you will hear the base unit speak "Learn Mode" from the speaker);
- Enter "1 2 3 4 #" into the telephone keypad (you will hear the base unit speak the numbers 1, 2, 3, 4 and # as you press the buttons);
- Enter first contact number and press the # (you will hear the base unit speak the numbers as you press them);
- Enter second number followed by # (you will hear the base unit speak the numbers as you press them) if you do not have a second number, press#;
- Enter third number followed by # (you will hear the base unit speak the numbers as you press them) if you do not have a third number, press#;
- Enter fourth number followed by # (you will hear the base unit speak the numbers as you press them) if you do not have a fourth number, press#;
- You will hear voice prompt "Programming Complete" (if you do not hear "Programming Complete", you must start over at the beginning of this section again);
- Slide switch back to desired location of Friends Only, Friends and 911 or 911 Only.

For set-up assistance, to order extra pendants or wall communicators, call (615) 562-0043