


User's Guide for the SkyAngel 911FD

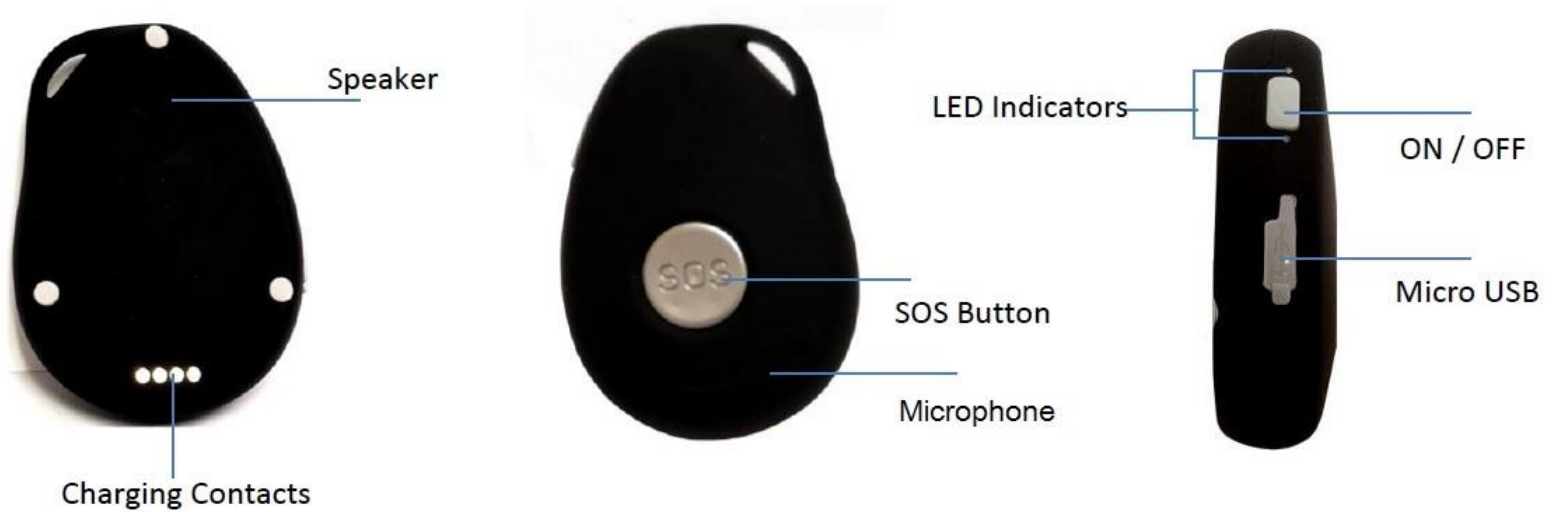

DEVICE CHARGING:

For the first time of use, please fully charge the unit for at least 2~3 hours

Using the USB charger or using docking station to charge

- Place the device on the docking station.
- Connect the Micro USB side of the cable to the docking station port and connect the other end of the cable to the designated AC power source (USB/AC adaptor).

SWITCHING THE DEVICE ON AND OFF:

- **To turn on the device:** press the side power button for 1 second, all the LEDs will flash rapidly. Device can be also turned on automatically by charging via USB or put it into the docking station.
- **To turn off the device:** press and hold the side button and SOS button together for 1 second until the LEDs off.

WATERPROOF:

Your SkyAngel 911FD is 100% waterproof and can be briefly worn in the shower or even submerged under water for short periods of time. Do not allow the device to be soaked for prolonged periods of time.

PLEASE NOTE: Do not remove the back cover at any time; this will break the waterproof seal. Also, removing the back cover voids any and all warranties.

WHAT DO THE FLASHING LIGHTS ON THE SIDE OF THE UNIT MEAN?

Charging Status – Blue Light:

- Blue light is on solid: the unit is charging
- Blue light is blinking quickly: unit has less than 15% power and should be recharged
- Blue light very rarely flashes: unit is fully charged and has GPS satellite signal

Cellular Status – Green Light:

- Green light single flash every 3 seconds: unit is connected to a cellular network
- Green light very rapidly flashing: unit is contacting 911

TESTING THE SYSTEM:

We recommend that you test your system as soon as possible. Press and hold the SOS button for 3 seconds until the device vibrates, and then the green light will start to flash rapidly to confirm the unit is about to make a call. After that, the unit will call 911. Once 911 answers, do not hang-up, inform 911 this is only a test. To end the call, press the SOS button. We highly recommend that you test your unit monthly to ensure the system is functioning properly and will be available if you need it.

ACTIVATING AN SOS ALARM:

Press and hold the SOS button for 3 seconds until the device vibrates, and then the green light will start to flash rapidly to confirm the unit is about to make a call. After that, the unit will call 911. If for some reason 911 fails to answer, the unit will start to dial the number again after 5 minutes. This will continue until the call is answered. To end the call, press the SOS button.

AUTOMATIC FALL DETECTION:

When the unit detects a fall from at least 2 feet above the ground it will begin to beep. This is to indicate that it has detected a fall and will soon begin to call 911 automatically. After approximately 10 beeps it will call 911. To cancel the call before it calls 911, press the SOS button.

GPS LOCATING:

The precision that 911 will be able to locate the unit can range from several feet to hundreds of feet. This unit has full two-way voice communication and that should be the primary method to direct assistance to the user.

LOW BATTERY ALARM:

The SkyAngel 911FD has a Low Battery Alarm. When the battery on the unit gets to less than 15%, the blue light on the unit will flash rapidly to warn the user that the unit needs to be charged.

For questions or technical assistance, please contact our office:



Assistive Technology Services

(615) 562-0043

Questions@AtsCares.com

NOTE: Our hours of operation are Monday – Friday, 8:00 AM – 6:00 PM Central Time

V170525