

SkyAngel *Classic*™

User Guide



Charging:

Use the **Magnetic Charging Cable** to charge the unit for **3 hours** initially to allow the unit battery to reach 100%.

If the unit is turned ON, when you attach the **Magnetic Charging Cable** the unit will speak "**Charger is connected**".

If the unit is turned ON, when the **Magnetic Charging Cable** is removed the unit will speak "**Charger is removed**".

The **front light** will turn **Red** when charging. The front light will turn **Green** when fully charged.

Press the **Red SOS button** once quickly and you will hear the unit speak the **Battery Charger Percentage**. The unit will speak "**Battery is xx**" xx represents the percentage of charge.

The **battery** can last up to **4 days** at full charge.

For **the best results**, charge about **30 minutes every day** to ensure the unit is always ready to use.

Power ON – Power OFF:

To turn the unit ON.

If the unit is off, **press and hold** the **Side Button** or the **Red SOS Button** for **15 seconds** to power the unit on. You will hear the unit play a chime sound.

As the unit is turning on you will see the **Front Light** turn **Red**, then **Green**.

When the unit has completely turned on and fully connected to a cell tower the **Front Light** will slowly **flash Red** on for 1 second and off for 2 seconds.

To turn the unit OFF.

If the unit is on, **press and hold** the **Green button** for 6 seconds or until you hear a chime. The unit will turn off and no lights will be flashing.

SOS Alert Functions:

This unit has two ways to activate the SOS function and call 911:

Red SOS Button:

When the unit is powered on, press and hold the **Red SOS Button** or the **Side Button** for 3 seconds. You will feel the unit vibrate and you will hear the unit speak "**Sos starting**". The unit will then call 911. Once 911 answers you can communicate through your unit to the 911 operator and tell 911 your situation. (911 will not automatically send help unless you verbally tell them where you are located and the situation)

Automatic Fall Detection:

If the unit **detects an injury causing fall of 2 feet or more above the ground**, you will hear the **unit speak "Sos starting"** then the unit will automatically call 911. You do not need to press any buttons. The call will be placed automatically. Once 911 answers you can communicate through your unit to the 911 operator and tell 911 your situation. (911 will not automatically send help unless you verbally tell them where you are located and the situation)

Cancel a Sos Call:

Press and hold the **Green button** for 6 seconds or until you hear a chime. The unit will turn off. Turn the unit back on to start using it again.

****Please allow the unit up to 2 minutes to connect a call if the cellular network is busy****

Green Button Health Features:

Keep in mind to achieve more accurate results the American Medical Association recommends:

Sit in a supported chair with back straight, feet flat on the floor, and legs uncrossed for at least 5 minutes before the measurement. Your arm should be supported on a flat surface at heart level. You may hold your watch hand over your chest with your other hand.

Pulse:

Press the **Green** Button **Twice** Quickly:

You will hear **"Testing pulse please wait a moment"**, in about 1 minute and 30 seconds you will hear the result as Example: **"Pulse 60 High Pressure 120 Low Pressure 73"**

Oxygen or SP02:

Press the **Green** Button **3 Times** Quickly:

You will hear **"Testing SpO2 please wait a moment"**, in about 1 minute you will hear the result as Example: **"SpO2 97%"**

Temperature:

Press the **Green** Button **4 Times** Quickly:

You will hear **"Testing body temperature please wait a moment"**, in about 30 seconds you will hear the result as Example: **"The body temperature is 87 point 3"**

Cancel a Health Measurement during the testing:

Press the **Side Button** quickly and release to cancel the measurement. The unit will speak **"Cancel Measurement"**.

Other:

Battery Level:

Press the **Red Sos Button** once quickly and the unit will speak **"Battery is 90"**. 90 represents the percentage of battery level.

Waterproof:

You can shower or take a bath with this unit for up to 30 minutes. After 30 minutes of submersion please wait 24 hours before re-submersing. It is IP67 certified waterproof.

Wrist Band is replaceable:

The wrist band size is 19 millimeters or $\frac{3}{4}$ inch. You can easily replace the band with another strap design by removing the pins and replacing the band.

Cellular:

This unit uses all available **3G** and **4G** cellular towers and carriers.

Size and Weight:

Length of unit from top to bottom: (Including band) 9.5 inches

Width unit face: **1 $\frac{1}{4}$ inches** - **Top to bottom** of face: **1 $\frac{7}{8}$ inches**

Thickness of unit: **5/8 inch** - **Weight: 1.6 ounces**

Battery replacement:

The battery is 600mah. The battery has a 1-year warranty from the factory. If the battery needs replacing outside the warranty, contact a certified battery replacement specialist or contact ATSCares.com.

Device Tag:

A device tag as seen below is attached to the watch. Please cut the clear plastic loop to remove. Please retain the tag in the event you would want to upgrade the watch in the future you will need the information on the tag.

Example Tag

**Warranty:**

This product is protected by a one-year manufacturer defect warranty. Please contact AtsCares.com to activate warranty coverage. Items covered under warranty are to be shipped to the warranty center paid by customer. The Item will be repaired or replaced under warranty. Warranty center will pay for return shipping in the U.S. Coverage outside the U.S., all shipping costs will be paid by customer.



SkyAngelClassic.com

Scan this QR code with your camera for information on your SkyAngelClassic