

SkyAngelSPORT

by: AtsCares.com

User Guide



Charging:

Use the **Magnetic Charging Cable** to charge the unit for **3 hours** initially to allow the unit battery to reach 100%.

If the watch is on and you attach the **Magnetic Charging Cable** the unit will display **“Screen Locked”** and you will see a battery indicator at the top right of the screen.

If the watch is off and you attach the **Magnetic Charging Cable** the SkyAngelSPORT screen will display briefly and you will see a battery indicator in the middle of the screen. This does not turn the unit on.

The **battery** can last up to **4 days** at full charge.

For **the best results**, charge about **30 minutes every day** to ensure the unit is always ready to use. There is no danger in leaving the unit charged for long periods of time.

Power ON – Power OFF:

To turn the unit ON.

If unit is off, **press and hold** the **Flat Side Button** for about **5 seconds** to power the unit on. You will hear the unit play a chime sound and the SkyAngelSPORT screen will display.

When the unit has completely turned on you will see the clock display that you have selected for a few seconds and then the screen will go blank based on the ‘sleep’ setting you have selected (15 seconds is the default sleep setting).

To turn the unit OFF.

If the unit is on,

1. **press the Flat Side Button** for 1 second to wake the screen from sleeping,
2. then **press and hold** the **Flat Side Button** for about 5 seconds or until you see the screen that says **“Turn Off Are you sure to switch off?”**
3. Click the Green Check to turn the watch off.
4. You will see the SkyAngelSPORT screen again and then the screen will go blank.

SOS Alert Functions:

This unit has two ways to activate the SOS function and call 911:

Round SOS Button:

When the unit is powered on, press and hold the **Round SOS Button** for 1 second. The unit will then call 911. Once 911 answers you can speak through your unit to the 911 operator and tell 911 your situation. (911 will not automatically send help unless you verbally tell them where you are located and the situation)

Cancel a Sos Call:

Press the **SOS button** to hang-up and cancel a call.

Automatic Fall Detection:

If the unit **detects an injury causing fall,**

1. you will hear the **unit Chime** and display a box that say **“A fall is detected if no fall, please cancel”**
2. then the unit will automatically call 911.

You do not need to press any buttons. The call will be placed automatically. Once 911 answers you can communicate through your unit to the 911 operator and tell 911 your situation. (911 will not automatically send help unless you verbally tell them where you are located and the situation)

Cancel a Fall Detection Call:

Press the **Red Cancel button** to hang-up and cancel a call.

****Please allow up to 2 minutes to connect a call if the cellular network is busy****

Swipe to the left Health Features:

Phone:

On this screen you can click the Phone icon and a keypad will appear. With the free version you can only dial 911. The subscription version will allow dialing of any phone numbers.

Pulse and Blood Pressure:

Press the **Red Heart** and then press **Start**:

You will see the **heart pulse** and within about 30 seconds you will see **Heart Rate** and **Blood Pressure** results.

Keep in mind to achieve more accurate results the American Medical Association recommends:

Sit in a supported chair with back straight, feet flat on the floor, and legs uncrossed for at least 5 minutes before the measurement. Your arm should be supported on a flat surface at heart level. You may hold your watch hand over your chest with your other hand.

Steps:

Press to see your steps today. Your steps are counted automatically and reset at midnight each day.

Alarm:

This feature is only available with a subscription.

To **Cancel a Health Measurement** during the testing:
Press the **Flat Side Button**

Settings:

Dial gallery – Scroll to view, then Click to select different clock faces

Volume – click + to increase volume and – to decrease volume

Brightness – click + to increase brightness and – to decrease brightness

Date and time – click to adjust date and time

Sleep – click to select 15 seconds, 30 seconds or 1 minute sleep timer

VoLTE – this should always be **Enabled**

Language – click to change the language

Turn off – click to select then click the **Green check** to turn off the watch

Information – click to see the system software version and IMEI number of the watch

Contacts:

This feature is only available with a subscription.

Temperature: - click to see your body **temperature**. History is only available with a subscription

QR Code: - click to see the **Registration code** and **APP code** for the subscription version

Swipe Up - to see the Step counter for the day:



Swipe Down - Battery Level, Signal Level, Date and Time:

If screen is Off (press **Flat Side Button** to wake up screen)

From any screen **Swipe Down**



You will see Grey **Cellular Signal Bars**, **Pin**, **No Sim Indicator**, **Battery Power Indicator**, **Date and Time** at top and **Brightness**, **Sound** and **Settings** button at the bottom

Waterproof:

You can shower or take a bath with this unit for up to 30 minutes. After 30 minutes of submersion please wait 24 hours before re-submersing. It is IP67 certified waterproof.

Wrist Band is replaceable:

The wrist band size is 20 millimeters or .78 inch. You can easily replace the band with another strap design by removing the pins and replacing the band.

Optional Upgrade - Subscription Features:

These are extra features that are only available with a subscription.

1. **Calls up to 3 phone numbers** when SOS is activated
2. Send **unlimited APP notifications** when the SOS is activated
3. Send **Text messages** to emergency contacts when SOS activated
4. **Manually Call numbers** from watch keypad
5. **Receive unlimited incoming phone calls to the watch**
6. **History of health results** viewable and automatic measurements
7. Ability to **turn on and off functions**
8. **Adjustable fall detection** sensitivity
9. **Review history** of Fall Detections and SOS activity
10. **Review history** of call log
11. Set 3 **Daily Alarms**
12. **“Find my watch”** function
13. Ability to receive **‘Low Battery’** alerts
14. **Block Unknown calls** to the watch
15. Ability to **track the current location of the watch** on a map
16. Ability to **track ‘History’** of the watch movement
17. Ability to set **‘Safe Zones’** on a map and receive **notifications** when the watch **arrives** or **leaves** the area.

For information on upgrading go to **SkyAngelSportUpgrade.com** or scan the QR code below.



SkyAngelSportUpgrade.com

Cellular:

This unit uses all available **3G** and **4G** cellular towers and carriers. If there is no cellular coverage from any carrier in your area this unit will not call 911.

Size and Weight:

Length of unit from top to bottom: (Including band) **10.4** inches

Width unit face: **1.5** inches

Top to bottom of face: **1.8** inches

Thickness of unit: **1/2** inch

Weight: **1.6** ounces

Battery replacement:

The battery is 600mah. The battery has a 1-year warranty from the factory. If the battery needs replacing outside the warranty, contact a certified battery replacement specialist or contact ATSCares.com.

Warranty:

This product is protected by a one-year manufacturer defect warranty. Please contact ATsCares.com to activate warranty coverage. Items covered under warranty are to be shipped to the warranty center paid by customer. The items will be repaired or replaced under warranty. Warranty center will pay for return shipping in the U.S. Coverage outside the U.S., all shipping costs will be paid by customer.



SkyAngelSPORT.com

Scan this QR code with your camera for information