

SteadyCane.com



Installing the batteries:

The SteadyCane requires 3 AAA batteries (not included) to power the LED light.


Open the battery compartment by pushing up on the end of the handle as shown below



Install 3 AAA batteries as shown below:



The height of the SteadyCane can be adjusted by pressing the spring loaded button near the top of the cane as shown below:

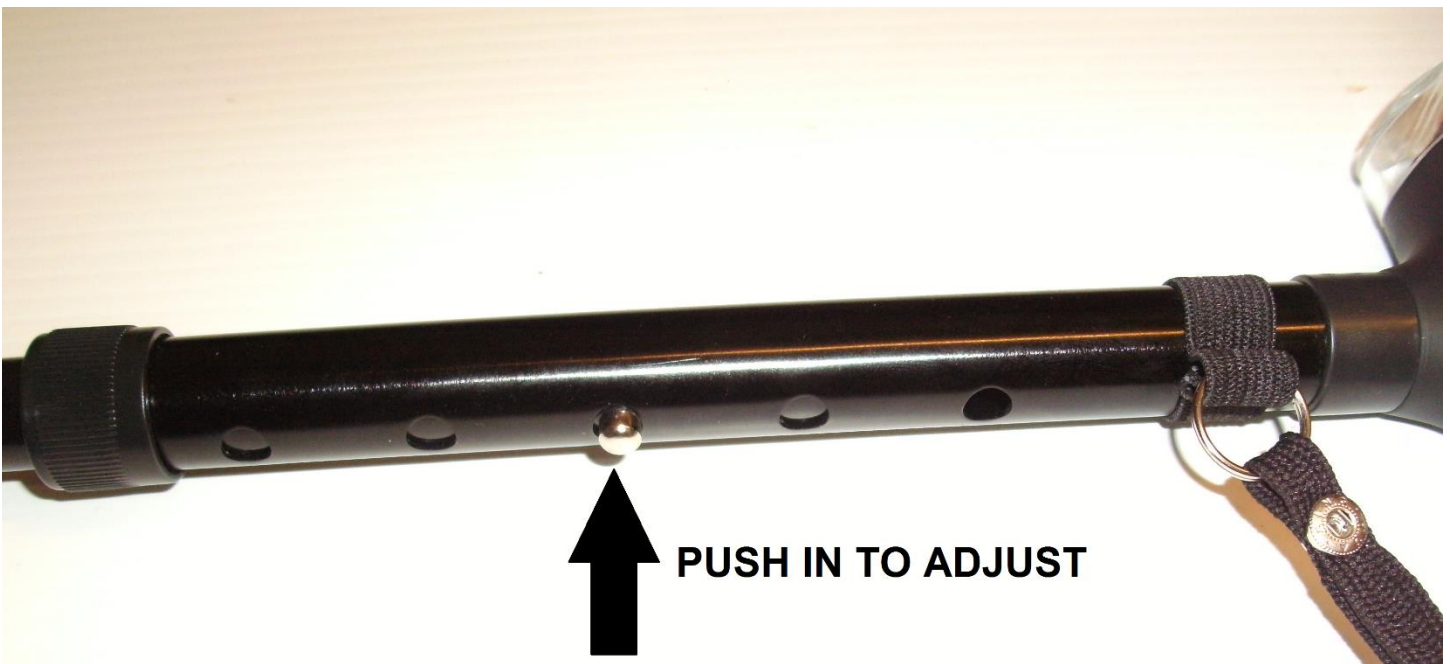

For questions or technical assistance, please contact our office:



Assistive Technology Services (615) 562-0043

Questions@ATS-TN.com

NOTE: Our hours of operation are Monday – Friday, 8:00 AM – 6:00 PM Central Time

V160716